

AQUA MARIS® CLEAN

100% NATURAL NASAL SPRAY

NOSE SPRAY 50 ML, 125 ML I 150 ML

Isotonic seawater, rich in salts and oligoelements, rinses away germs and impurities and promotes a clear passage of the upper airways.

COMPOSITION

Solution contains Adriatic sea water rich in salts and oligoelements. Oligoelements in the solution have an important role in maintaining good function of the nasal mucosa, which helps the performance of the upper respiratory tract.

PRESERVATIVE FREE.

PACKAGING

50 ml solution in a bottle with a microspray.

125 ml solution in a bottle with a microspray.

150 ml solution in a bottle with a microspray.

PURPOSE

Normal breathing depends on the state of the nasal mucosa. The accumulation of dust particles, allergens, germs on the nasal mucosa inhibits its natural function. Colds, allergies, flu and many other conditions cause increased secretion, swelling of nasal mucosa and a build-up of secretion, which can hinder breathing. Seawater has a beneficial effect on the airway mucosa because of its mineral salt content.

WHEN AND HOW TO USE AQUA MARIS® CLEAN NASAL SPRAY

Aqua Maris® Clean, with its specially-formed applicator, removes secretion, dust particles, germs and allergens from the nose. It is recommended for people with colds and allergies, and people exposed to dust particles and other impurities.

Spray Aqua Maris® Clean 1-2 times in each nostril, several times a day (2-6 times), and more often if necessary. Wash and dry the applicator after each use.

Due to its gentle nature, the product is also suitable for everyday use to help maintain clear nasal passages.

Aqua Maris® Clean has a gentle effect that supports the physiological condition of the mucosa and has no systemic effects. It is therefore suitable for everyone, including children, pregnant women and nursing mothers.

For hygiene reasons, the product should only be used by one person; do not share it with others.

Why not try other Aqua Maris® products?

Aqua Maris®. Sharing the natural power of Adriatic sea water. For everyone. Everyday.

PRODUCT NAME	PURPOSE	INGREDIENTS
Aqua Maris® Baby nasal drops	moisturise and clean the nose; suitable from baby's first day of life	isotonic seawater
Aqua Maris® Baby nasal spray	unblocks the nose and removes impurities, germs from baby's nose	isotonic seawater
Aqua Maris® Classic nasal spray	moisturises and cleans the nose	isotonic seawater
Aqua Maris® Refresh nasal spray	refreshes, protects and cleans	seawater + lavender and immortelle hydrolates
Aqua Maris® Plus nasal spray	regenerates the nasal mucosa	seawater + d-panthenol
Aqua Maris® 4Allergy nasal spray	relieves allergy symptoms	Ectoin® 2% + sea salt
Aqua Maris® Strong nasal spray	naturally decongests the nose	hypertonic seawater
Aqua Maris® Thalassotherapy nasal and sinus wash	washes the nasal cavity and sinuses with a solution containing beneficial natural sea salt and medicinal herbs from the Mediterranean	neti pot + 30 sachets of natural sea salt without additives, enriched with myrtle and immortelle essential oils
Aqua Maris® Thalassotherapy sea salt with myrtle and immortelle essential oils	Adriatic sea salt for use with Aqua Maris® Thalassotherapy, a nasal and sinus wash	30 sachets of natural sea salt without additives, enriched with myrtle and immortelle essential oils

Aqua Maris® Grlo
throat and mouth spray

relieves irritation of the throat
and the oral cavity

hypertonic seawater

Aqua Maris® Propolis
throat and mouth spray

helps soothe irritated and i
nflamed throat and reduces i
rritation of the oral cavity

seawater + propolis extract

MANUFACTURER

JGL d.d., Svilno 20, 51000 Rijeka, Croatia

 0373

EXPLANATION OF PRODUCT SYMBOLS



Manufacturer



CE mark



Expiry date



See instructions for use



Batch number